

ABECCA

BY DESIGN

Furniture / Mattress Cleaning Guide

Abecca recommends weekly cleaning of furniture and mattresses, as keeping Vinyl, PVC & LimberShield™ products well maintained is essential to keep durability and visual appeal.

Make sure you attend to spills immediately, wipe spill using a clean, dampened cloth and if needed a mild solution of liquid detergent and warm water or an approved cleaning solution, repeat if necessary.


Please ensure all wet cleaning is followed by drying using a clean dry cloth.

The following products are safe and effective for use on Abecca®'s products:

- Whiteley V-Wipes Hospital Grade Disinfectant
- Clinell Universal Sanitising Wipes
- Reynard Neutral Detergent Wipes
- Whiteley Viraclean
- Virex II J-Flex E6 Disinfectant Cleaner and Deodorant
- Agar Ph7 Neutral Detergent
- Agar Kuranda

**Please follow these recommendations
to get the best use out of your furniture**

If you would like to use a different cleaning product, please contact Abecca® at customerservice@abeccadesign.com before use to confirm compatibility with our Vinyl, PVC & LimberShield™ thermo-fused technology, failure to do so may void your warranty.

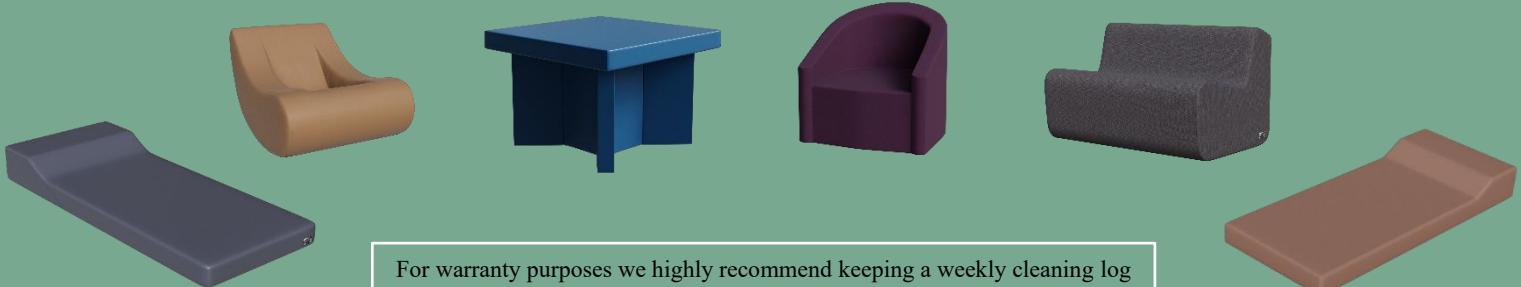

For warranty purposes we highly recommend keeping a weekly cleaning log as this will be asked for along with the products used to clean the furniture when any claim is made.

 **Armadillo®**

 **EAGLE RANGE™**

 **HAWK RANGE™**

Order enquiries please contact:

W: abeccadesign.com E: customerservice@abeccadesign.com

NZ p: +64 6 358 9642 | 40 Onehunga Mall, Onehunga, Auckland 1061, New Zealand

AU p: +61 2 8089 3950 | 10 Herb Elliot Ave, Sydney Olympic Park, NSW 2127, Australia